


 **Vitasure™**

**Absorbable
Hemostat**

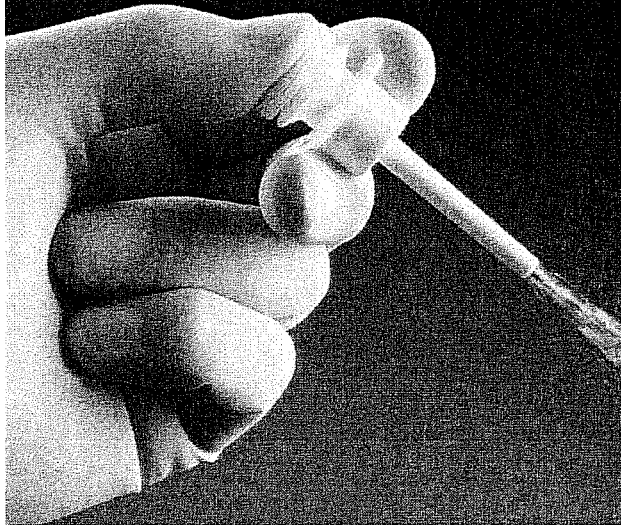

 **ORTHOVITA®**

Distributed by Orthovita, Inc., Malvern, PA 19355
© 2008 Medafor, Inc.
Made in U.S.A.
Manufactured under U.S. Patent 6,060,461.



**Absorbable Hemostat
Microporous Polysaccharide Hemosphere (MPH®) Technology**

DESCRIPTION

Vitasure is a medical device intended for application to surgical wound sites as an absorbable hemostat. This technology incorporates hydrophilic, flowable, microporous particles synthesized by cross-linking purified plant starch through a proprietary process; Vitasure is a 100% plant based polysaccharide. Vitasure contains no animal or human components. Vitasure is a fine, dry, sterilized white powder that is biocompatible, non-pyrogenic, and is typically absorbed within 24 to 48 hours.

ACTION

Vitasure particles are hydrophilic molecular sieves that enhance natural hemostasis by concentrating blood solids such as platelets, red blood cells, and blood proteins on the particle surfaces to form a gelled matrix. The concentrated gel matrix provides a barrier to further blood loss and is formed regardless of the patient's coagulation status. The concentration of clotting factors and platelets in the gel serves to enhance normal clotting reactions and creates stable hemostatic plugs. The absorption process begins immediately and is dependent on several factors, including the amount applied and site of use.

INDICATIONS

Vitasure is indicated in surgical procedures (except neurological and ophthalmic) as an adjunctive hemostatic device to assist when control of capillary, venous, and arteriolar bleeding by pressure, ligature, and other conventional procedures is ineffective or impractical.

CONTRAINDICATIONS

Do not inject or place Vitasure into blood vessels as potential for embolization and death may exist.

WARNINGS

Vitasure is not intended as a substitute for meticulous surgical technique and the proper application of ligatures or other conventional procedures for hemostasis.

Once hemostasis is achieved, excess Vitasure should be removed from the site of application by irrigation and aspiration particularly when used in and around foramina of bone, areas of bony confine, the spinal cord, and/or the optic nerve and chiasm. Vitasure swells to its maximum volume immediately upon contact with blood or other fluids. Dry, white Vitasure should be removed. The possibility of the product interfering with normal function and/or causing compression necrosis of surrounding tissues due to swelling is reduced by removal of excess dry material.

Safety and effectiveness of Vitasure have not been clinically evaluated in children and pregnant women. Because there have been reports of decreased amylase activity in newborns up to 10 months, absorption rates of Vitasure in this population may be longer than 48 hours.

Vitasure should be used with caution in the presence of infection or in contaminated areas of the body. If signs of infection or abscess develop where Vitasure has been applied, re-operation may be necessary in order to allow drainage.

Safety and effectiveness in neurosurgical and ophthalmic procedures has not been established.

Vitasure should not be used for controlling post-partum bleeding or menorrhagia.

PRECAUTIONS

When Vitasure is used in conjunction with autologous blood salvage circuits, carefully follow instructions in the Administration section regarding proper filtration and cell washing.

Vitasure is intended to be used in a dry state. Contact with saline or antibiotic solutions prior to achieving hemostasis will result in loss of hemostatic potential.

Vitasure is not recommended for the primary treatment of coagulation disorders.

No testing has been performed on the use of Vitasure on bone surfaces to which prosthetic materials are to be attached with adhesives and is therefore not recommended.

Vitasure is supplied as a sterile product and cannot be resterilized. Unused, open containers of Vitasure should be discarded.

Do not apply more than 50g of Vitasure in diabetic patients as it has been calculated that amounts in excess of 50g could affect the glucose load.

In urological procedures, Vitasure should not be left in the renal pelvis or ureters to eliminate the potential foci for calculus formation.

ADVERSE REACTIONS

In a randomized prospective, concurrently controlled clinical trial, a total of 288 randomized patients received Vitasure or the Control (Gelatin Sponge with or without Thrombin). The most common recorded adverse events were pain related to surgery, anemia, nausea, and lab values out of normal range. The following is a complete list of adverse events reported in greater than 5% of the Vitasure treated patients. The corresponding adverse events for the Control group are listed for comparison. None of the adverse events that occurred were judged by the Data Safety Monitoring Board to be related to the use of Vitasure.

ADVERSE EVENTS REPORTED IN GREATER THAN 5% OF THE VITASURE TREATED PATIENTS

Adverse Events	Vitasure	Control
Pain related to surgery	140 (96.6%)	139 (97.2%)
Anemia	52 (35.9%)	49 (34.3%)
Nausea	44 (30.3%)	39 (27.3%)
Lab values out of normal range	26 (17.9%)	20 (14.0%)
Arrhythmia	24 (16.6%)	31 (21.7%)
Constipation	27 (18.6%)	12 (8.4%)
Respiratory Dysfunction	19 (13.1%)	18 (12.6%)
Hypotension	18 (12.4%)	18 (12.6%)
Fever	12 (8.3%)	15 (10.5%)
Pruritis	11 (7.6%)	12 (8.4%)
Ecchymosis	13 (9.0%)	8 (5.6%)
Tachycardia	10 (6.9%)	10 (7.0%)
Edema	9 (6.2%)	9 (6.3%)
Pain unrelated to surgery	9 (6.2%)	8 (5.6%)
Hemorrhage	9 (6.2%)	7 (4.9%)
Hypertension	8 (5.5%)	7 (4.9%)

Other adverse events reported in fewer than 5% of the Vitasure population included: Paresthesia, Cutaneous Bleed, Infection, Seroma, Confusion, Renal Insufficiency, Heartburn, Diarrhea, Vertigo, Hypovolemia, Pneumonia, Pleural Effusion, Paresis, Dermal Irritation, Urinary Dysfunction, Muscle Spasms, Hematuria, Ileus, Coagulopathy, Pneumothorax, Dysphagia, Ischemia, Deep Vein Thrombosis, Gout, Inflammation, Necrosis, Hematoma, Hypothermia, Agitation, Rash, Hypoxaemia, Myocardial Infarction, Hyperthermia, Hypercapnia, Clostridium Difficile, Eye Irritation, Xerostomia, Nerve Palsy, Pericardial Effusion, Cardiac Tamponade, Excoriation, Fatigue, Flatus, Unrelated Illness, Cellulitis, Syncope, Shivering, Sore Throat, Alkalosis, Heel Ulcer, Anastomotic Leak, Clot, Gastritis, Left Ventricular Fistula, Liver Insufficiency, Adrenal Insufficiency.

ADVERSE REACTIONS THAT HAVE BEEN ATTRIBUTED TO OTHER ABSORBABLE HEMOSTATIC AGENTS

Note that Vitasure is a unique absorbable hemostatic agent consisting of 100% purified plant starch and it exhibits a faster absorption time (approximately 24 to 48 hours) compared to other absorbable hemostatic agents that absorb in 3 to 8 weeks. The following adverse events have been reported for other absorbable hemostatic agents and may apply to the use of Vitasure:

Paralysis and nerve damage have been reported when hemostatic agents are used in or in proximity to foramina in bone, areas of bony confine, the spinal cord, and/or the optic nerve and chiasm. While most of these reports have been in connection with laminectomy, reports of paralysis have also been received in connection with other procedures.

Compression of the brain and spinal cord resulting from the accumulation of sterile fluid has been observed.

CLINICAL STUDIES

Study Design and Objectives:

A prospective, multi-center, multi-specialty, randomized, non-inferiority, controlled clinical trial was conducted. Two hundred eighty-eight (288) patients were randomized and treated at nine investigational centers. The objective of the study was to evaluate the safety and effectiveness of Vitasure versus a commercially available absorbable gelatin sponge used with or without Thrombin to control intraoperative bleeding in orthopedic, general, and cardiac surgeries. Patients were randomized only after a lesion suitable for treatment with an adjunctive hemostatic device was identified. Complete hemostasis was defined as cessation of bleeding. The primary endpoint was complete hemostasis of the first treated lesion within 5 minutes (3 minutes for cardiac). Although multiple lesions in the same patient could be treated per the protocol, only the first treated lesion was used to determine the effectiveness as this was the only lesion that was truly randomized.

Primary Endpoint

For the primary endpoint, complete hemostasis of the first treated lesion within 5 minutes (3 minutes for cardiac) was achieved in 90.3% of the randomized and treated subjects in the Vitasure group and in 80.4% of the randomized and treated subjects in the control group. Vitasure was demonstrated to be non-inferior to the control ($p < 0.0001$). The upper 95% limit on the difference in proportions (Control – Vitasure) was less than zero (-2.4%).

Primary endpoint data for the entire randomized and treated patient population as well as stratified by surgical specialty are summarized in the table below:

COMPLETE HEMOSTASIS OF FIRST TREATED LESION WITHIN 5 MINUTES (3 FOR CARDIAC)

Primary Efficacy Endpoint	Vitasure n/N (%)	Control n/N (%)
Hemostasis of First Treated Lesion	131/145 (90.3%)	115/143 (80.4%)
Surgical Application	Vitasure n/N (%)	Control n/N (%)
General (within 5 minutes)	68/72 (94.4%)	56/72 (77.8%)
Orthopedic (within 5 minutes)	33/35 (94.3%)	32/37 (86.5%)
Cardiac (within 3 minutes)	30/38 (78.9%)	27/34 (79.4%)

Secondary Endpoint

A secondary endpoint was time to hemostasis for the first treated lesion. The data for time to hemostasis are summarized below. The times to achieve complete hemostasis for the Vitasure and Control groups were statistically different using the chi square test ($p = 0.003$).

**CUMULATIVE PERCENT OF PATIENTS WITH COMPLETE HEMOSTASIS
FIRST TREATED LESION**

Time to Complete Hemostasis	Vitasure n/N (%)	Control n/N (%)
1 minute	73/145 (50.3%)	47/143 (32.9%)
2 minutes	96/145 (66.2%)	83/143 (58.0%)
3 minutes (Cardiac to 3 minutes only)	124/145 (85.5%)	103/143 (72.0%)
4 minutes	130/145 (89.6%)	111/143 (77.6%)
5 minutes	131/145 (90.3%)	115/143 (80.4%)

When data are stratified by surgical specialty, the median times to hemostasis were shorter for the Vitasure group than for the Control group. The median times are summarized in the table below. The median time to hemostasis for the Vitasure arm was statistically different from the Control arm using the non-parametric Wilcoxon sign-rank test ($p=0.002$). Note that one subject in the Vitasure arm had a time to complete hemostasis of 56.8 minutes and this value is considered an outlier.

COMPARISON OF TIME TO HEMOSTASIS – FIRST TREATED LESION

	Vitasure Median (n) (min, max)	Control Median (n) (min, max)
Median Time to Hemostasis (minutes)	1.0 (144) (1.0, 56.8)	2.0 (143) (1.0, 19.2)
General	2.0 (72) (1.0, 19.5)	2.0 (72) (1.0, 15.0)
Cardiac	2.0 (38) (1.0, 56.8)	3.5 (34) (1.0, 19.2)
Orthopedic	1.0 (34) (1.0, 6.2)	2.0 (37) (1.0, 7.0)

DIRECTIONS FOR USE

Inspect the integrity of the Vitasure packaging and applicator prior to use. If either is damaged, do not use.

Remove the applicator cap using a bending and twisting motion. For maximum benefit the following technique is recommended:

1. Blot, wipe, or suction the bleeding tissue. **It is important to remove excess blood so Vitasure may be applied immediately and directly to the site of active bleeding.**
2. Immediately apply a liberal amount of Vitasure at the site of bleeding within the wound, as close to the source of bleeding as possible, completely covering the wound. Deep wounds may require equally deep application of Vitasure. To minimize occlusion of the tip, pressure should be applied to deliver Vitasure as the applicator enters the wound.
3. Quickly apply wound-appropriate, direct pressure over the treated site; use of a non-adhering substrate to apply pressure may prevent adhesion of the formed clot to the surgical glove or other instrumentation. Amount and duration of pressure is wound dependent. For oozing, pressure may not be necessary. For more profuse bleeding wounds, pressure should be maintained longer.

4. If bleeding or oozing continues, remove excess Vitasure and reapply.
5. If any material (ie, surgical glove, sponge, etc) adheres to the newly formed clot, irrigate the material with saline and carefully remove it from the treated site.
6. Immediately upon contact with fluid, Vitasure will swell to approximately 5 times its original volume. Once hemostasis is achieved, excess Vitasure should be carefully removed by irrigation and aspiration.

ADMINISTRATION

Aseptic technique should always be used. A liberal amount of Vitasure should be applied to the bleeding site (see DIRECTIONS FOR USE) followed by pressure until hemostasis is achieved. After hemostasis is achieved, Vitasure should be removed by irrigation and/or aspiration. Opened containers of Vitasure should always be discarded.

Animal studies have indicated that Vitasure does not enhance infection of the wound site. Vitasure is rapidly degraded and cleared from the treatment site by endogenous alpha amylases. In an *in vivo* rat infection model, a specified amount of *E. coli* was inoculated into an induced abdominal wound followed by treatment with control (no hemostatic agent), positive control (absorbable gelatin sponge), and Vitasure. After 72 hours, animals were sacrificed and microbiology homogenate culture of the tissue removed found that there is no increase in the rate of *E. coli* infection in wounds treated with Vitasure [an average of 6.9×10^5 colony forming units (cfu) found after 72 hours] compared to the rate in wounds treated with no hemostatic agent (control) [average of 3×10^5 cfu after 72 hours] indicating that Vitasure is not a nidus for infection. The rate of infection in wounds treated with gelatin sponge [average of 2.4×10^7 cfu after 72 hours] was found to be significantly enhanced versus both the control and Vitasure.

When Vitasure is used in conjunction with autologous blood salvage circuits, a 40 μ cardiotomy reservoir, cell washing, and 40 μ transfusion filter, such as a LipiGuard™ or Pall filters, must be used.

HOW SUPPLIED

Vitasure is supplied in bellows applicators containing 3grams. In the Vitasure Trial, subjects in the Vitasure arm could have up to five (5) bleeding lesions treated with Vitasure. In the trial, surgeons used between approximately 1 to 9 g to treat these lesions. In the Vitasure Trial, the mean number of product applications onto the first treated lesion was 1.4 (range 1 to 5) and for control, the mean number of product applications was 1.4 (range 1 to 5).


STORAGE AND HANDLING

Vitasure should be stored to avoid temperature extremes (less than -40°F (-40°C) and greater than 140°F (60°C)). Once the applicator has been opened, contents are subject to contamination. It is recommended that Vitasure be used as soon as the applicator is opened and unused contents discarded.


Disclaimer: Neither Medafor, Inc., nor Orthovita, Inc. shall be liable for any direct, incidental, or consequential damages other than as specifically provided under applicable law or contract. No individual has the authority to bind Medafor, Inc. or Orthovita, Inc. to any representation or claim except as specifically set forth herein. Descriptions or specifications are presented to generally describe Vitasure and related procedures, and do not constitute any expressed claims or practices.

Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician or properly licensed practitioner.

DEFINITION OF SYMBOLS

 Latex Free

 Rx Only

Distributed by:



Orthovita®, Inc.
Malvern, PA 19355 USA
PHONE: 610-640-1775
FAX: 610-640-1714
www.orthovita.com

Orthovita® Europe
Interleuvenlaan 5
BELGIUM
PHONE: +(32) 16 39 2890
FAX: +(32) 16 39 2891



Manufactured in USA by:
Medafor, Inc.
Minneapolis, MN 55430 USA
PHONE: 763-571-6300
FAX: 763-571-1035
www.medafor.com

 SCANLAN GROUP B.V.
Postbus 75664
1118 ZS Schiphol-Tripport
The Netherlands
PHONE: 31 20 653 0553
FAX: 31 20 653 3053

© 2008 Medafor, Inc. All rights reserved. Printed in the USA.
Vitasure is manufactured under US Patent #6,060,461. Additional Patents Pending.

LIT-0070 Rev C
09/08